




D60 T60 T45


tethys - tethys - tethys

INSTRUCTION SHEET


SELECTING THE WASHING PROGRAMME

	Switch on the device by pressing the On/Off key (hold down for 2 seconds). Close the door by shutting it against the appliance until the lock clicks shut. The buttons are only enabled with the door closed.
	Use the Selection buttons to select the program.
	Press the Start/Stop button to start the cycle (hold it down for 2 seconds). Once the selected program has been successfully completed, the display shows "End".

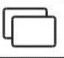
INTERRUPTING THE CYCLE

	A cycle in progress can be interrupted by pressing the Start/Stop button. It is not possible to open the door. - If the button is pressed again within 60 seconds – the cycle is re-started. - If the button isn't pressed within 60 seconds: an automatic reset cycle is performed automatically to return the machine to the standby condition.
---	--


DOOR OPENING

	Press the Door opening button: the appliance gives a <i>beep</i> when it receives the signal. Release the button and the door will open automatically after a few moments.
---	---

RESET PROCEDURE

	Press the Functions and Start/Stop buttons and hold them down for about 2 seconds, until a beep is heard. "P" appears on the display and the RESET procedure starts.
--	--

ALARMS

LED	MEANING
	When an alarm occurs: Led "Warning" on and display indication. "AF" – fatal alarm. "A-" – non fatal alarm. If a non-fatal alarm occurs you can press the Start/Stop button to continue using the device. When an error occurs, generally the device manages the anomaly itself: in this case the alarm code flashes on the display until the problem has been dealt with: no reset commands are accepted during this time. Any RESET forced by the user will not be accepted until the problem has been dealt with and the alarm code on the display is "steady".

PROGRAMS

[Note: drying phase only on Tethys D60]

1 Auto-disinfection 93°C (empty chamber)	9 Thermodisinfection 93°C - 10' [+ Drying]
2 Prewash	10 Custom (90°C - 1')
3 Washing 60°C - 1' [+ Drying]	11 Custom (93°C - 3')
4 Washing – chemical agent P3 - 80°C - 1' [+ Drying]	12 Custom (93°C - 10')
5 Thermodisinfection 80°C - 10' [+ Drying]	13 Custom (93°C - 1')
6 Thermodisinfection 90°C - 1' [+ Drying]	14 Custom (93°C - 5')
7 Thermodisinfection 90°C - 5' [+ Drying]	15 Custom (Auto-disinfection 93°C – 1')
8 Thermodisinfection 93°C - 5' [+ Drying]	16 Service

Note: "PdiS" self-disinfection cycle. Run the program without load, no load inside the washing chamber.

THIS DOCUMENT

Instruction sheet for basic operations.

For all the details concerning the correct and safe use of the devices in question, for the updated program table, for the interpretation of alarm messages and for the requirements for installation: Refer to the user manual and program table - full versions.